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Restores healthy gums and soothes inflamed and traumatised oral tissues

GENGIGEL® is a medical device containing high molecular weight (HMW) hyaluronic acid (in the form of sodium hyaluronate), obtained by a biotechnological manufacturing process which guarantees a high level of purity. Its formulation is specifically created in order to resemble that of the hyaluronic acid found in normal gingival tissue.

Hyaluronic acid is a naturally occurring physiological constituent of connective tissue (especially in the gingival mucosa), where it performs anti-oedematous and tissue repair functions. The physico-chemical and macro-aggregating properties of hyaluronic acid help to explain its anti-inflammatory properties.

HMW hyaluronic acid is found in connective tissue throughout the body in a selective and specific manner. It is particularly concentrated in the outermost layers of healthy gingival tissue, where it contributes to both the barrier function and tensile strength of the periodontal ligament. The presence of HMW hyaluronic acid is essential for the maintenance of healthy gingival tissue.

In periodontal disease and other tissue traumas (inflammatory conditions, gingival pockets, wounds, etc.) the tissue requirement for hyaluronic acid increases considerably (by up to some 200% compared with the basal values), thus demonstrating its specific role in regulating cell turnover and optimising local tissue regeneration.

The hyaluronic acid deficiency that arises under these conditions prevents the tissue from:

- restoring the normal free water balance (as a consequence of which swelling and compression causes an increase in pain);
- activating the migration of fibrocytes (cells contributing to tissue healing), as a consequence of which healing is delayed.

The lack of suitable HMW hyaluronic acid is therefore responsible for the continuation of the inflammatory condition.

Studies in man show that the availability of exogenous hyaluronic acid with similar characteristics to those of the naturally occurring component helps to facilitate the natural tissue healing and recovery process.

GENGIGEL® is therefore an innovative product providing long-acting biotechnologically manufactured hyaluronic acid with high molecular weight. The application of hyaluronic acid provides a periodontal tissue/fluid balance with accelerated healing and repair functions by preventing the deficiency of natural gingival HMW hyaluronic acid and enhancing its effects. Hyaluronic acid does not develop any local or systemic pharmacological effects on the mucosa. Hyaluronic acid is extensively depolymerised and reduced to its basic sub-unit constituents by plaque bacterial enzyme action.

GENGIGEL® provides maximum adhesion and thus allows hyaluronic acid (which would otherwise be eliminated by constant salivary drainage) to remain in situ.

GENGIGEL® is extremely safe, presenting no contraindications or side effects, and can be used without restriction by children, pregnant women and the elderly. The safety profile of GENGIGEL® is particularly useful when anti-inflammatories are contraindicated. By virtue of the properties of Xylitol, a natural sweetener that does not promote tooth decay or cause significant variations in the blood sugar level, the product is also suitable for diabetics. It does not have any known drug interactions, so it may be safely used during specific therapies e.g. with antibiotics.

Indications: inflammatory conditions (gingivitis, gingival bleeding, gingival retraction, gingival pockets) and other traumatic conditions (in particular mouth ulcers, wounds caused by removable prostheses or orthodontic appliances, cleaning, abrasions, tooth extractions, post-surgical recovery, etc.), and those disease conditions where the gingival mucosa requires increased concentrations of hyaluronic acid.

Instructions for use: after correct oral hygiene, apply GENGIGEL® three to five times a day (after main meals) for 3-4 weeks, continuing until all symptoms have disappeared. It is advisable not to eat or drink for at least 30 minutes after application.

Ge: apply a film of gel onto the gingival tissue, massaging gently with clean fingers to aid its correct distribution on the mucosal surface.

Hydrogel: rinse with 10 ml (one dose) for 1-2 minutes. The compound is ready for use; do not dilute.

Spray: each puff guarantees homogeneous, constant distribution of the liquid on the mouth surface; 1 or 2 puffs on inflammatory or traumatised tissue. Keep the bottle vertical.

Harmless if swallowed.

Composition

Principal ingredient: Sodium Hyaluronate

Other components:

Ge: Aqua, Xylitol, Cellulose Gum, Alcohol, PEG 40 Hydrogenated Castor Oil, PVA, Polycarbophil, Dichlorobenzyl Alcohol, Aroma, Sodium Hydroxide, Acid Blue 9 (CI 42090)

Hydrogel: Aqua, Xylitol, Alcohol, Polysorbitate 20, PEG 40 Hydrogenated Castor Oil, Dichlorobenzyl Alcohol, Polycarbophil, Aroma, Citric Acid, Sodium Hydroxide, Acid Blue 9 (CI 42090)

Spray: Aqua, Xylitol, Alcohol, PEG 40 Hydrogenated Castor Oil, Sodium phosphate, PVA, Aroma, Dichlorobenzyl Alcohol, Trisodium phosphate.

Contraindications: the use of GENGIGEL® is contraindicated in the event of hypersensitivity to any of its ingredients.

Precautions and warnings: do not use in case of any visible damage. Store at room temperature, protect against direct heat sources. Do not use after the expiry date. Keep out of children's reach. Should the symptoms persist, consult a healthcare professional.