## Eigenschappen

Tube van 15 ml, 60 ml of 140 ml.

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Tube of 15 ml, 60 ml or 140 ml.

## Indications

- Cutaneous use
- On first or second degree burns, sunburn and non-infected wounds
- In the case of reddened skin due to radiotherapy and lasertherapy

## Composition

Purified water, ethylene glycol stearate, liquid paraffin, pentanediol, stearic acid, paraffin wax, perhydrosqualene, avocado oil, sodium alginate, trolamine, cetyl palmitate, potassium sorbate, scent of honellia.

## Properties

Lamiderm® Repair's formula 2 in 1 contributes to wound healing and skin hydration. Lamiderm® Repair maintains a moist environment for skin regeneration.

- Lamiderm® Repair hydrates the wound.
- Lamiderm® Repair drains the wound.

Lamiderm® Repair prevents dehydration and supports the regeneration of the skin thanks to its adapted formulation which contains particularly moistening and moisturizing agents, including avocado oil. Lamiderm® Repair contributes to better wound healing. Lamiderm® Repair is a multifunctional and skin-friendly restorative emulsion. It does not contain parabens and can be applied to face and body. Lamiderm® Repair is suitable for the whole family; young and old, infants and pregnant women.


## Instructions for use

### Tube opening

At the first use, pierce the tube seal with top of the cap.

### - First degree burns and sunburn

Apply Lamiderm in a thick layer and allow it to penetrate without rubbing it in. Repeat this process two or four times a day.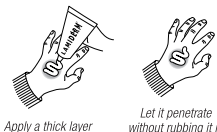

### - Second degree burns and non-infected wounds


After cleansing the burn or non-infected wound, apply a thick layer of Lamiderm and repeat this several times each day. If

required, cover with a moistened compress. Do not use a dry absorbent dressing.

### - Reddened skin due to radiotherapy and laser therapy

Apply Lamiderm two or three times a day, according to the physician recommendations, and let it penetrate with a light massage.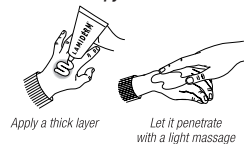

## Precautions for use

Do not use:

- If the wound is bleeding
- If the wound is infected

This medical device is not a sunscreen. Do not use in case of hypersensitivity to any component of the product. Keep out of the reach of children. Store away from heat and in the carton box. Do not exceed the use-by date clearly indicated on the packaging and the tube.

## Vigilance

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Revision date:** 06/2023

**Distributor:** Qualiphar N.V./S.A., Rijksweg 9, B-2880 Bornem

**Manufacturer:** Gifrer Barbezat

10 Avenue des Canuts  
69120 Vaulx-en-Velin FRANCE

30012014-14  
BEL-LAM-NOT/8-05/23